

Eye Ball Stand

DEVICE DESCRIPTION

An Eyeball stand consists of a stainless-steel base and a spring-loaded clamp. The base has a concave cavity for keeping the eyeball in place whereas the clamp is used to fix the eyeball firmly and regulate the intraocular pressure.

INTENDED USE/ INDICATION FOR USE

An Eyeball stand is used for micro-surgical training on Human / Animal eyes and/or cutting a full/partial thickness donor button from intact globe.

CONTRAINDICATIONS

Not Known

HOW SUPPLIED

Eyeball Stand is supplied un-sterile, in individual leather box. The overall packing contains Eyeball stand and medical literature insert, in an outer leather box, with external identification stickers.

CAUTION

For External Use only

⚠ WARNING


- Only qualified medical practitioner should use the device.
- Read instruction before use.
- Store at room temperature.
- For External use only.
- Keep out of reach of children.

PRECAUTION

- When handling the product care should be taken to avoid damage from handling.

DIRECTION FOR USE

- Loosen the screw knob (C), pull the clamp (B) upwards thereby creating space for keeping the eyeball.

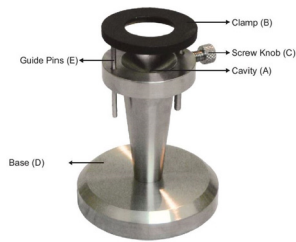

- Place the eyeball into the cavity (A) with cornea facing upwards.
- Press the spring-loaded clamp slightly on the eyeball. After centering the cornea in the ring, further press the clamp to get the desired Intraocular Pressure (IOP) and lock using the screw knob (C).
- Once the eyeball is properly fixed and desired IOP is achieved, proceed with intended surgical/ training procedure.

DISPOSAL OF THE USED DEVICE

The used device should be disposed off in compliance with the local regulatory requirements.

REUSE OF DEVICE

The device should undergo a detailed multi step process of cleaning & sterilization before reuse.

RETURN OF DAMAGED PRODUCT

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/ exchange.

EXPLICATION OF SYMBOL:

Symbol	Meaning
	Caution
	Manufacturer
	Date of Manufacture
	Keep away from sunlight
	Consult instructions for use
	Re-use
	Authorized Representative in the European Community
	This side up
	Do not use hook

Symbol	Meaning
	Conformity of European Norm
	Batch Code
	Temperature Limit
	Do not use if package is damaged
	Humidity Limitation
	Handle with care
	Maximum stacking
	Medical Device
	Non-Sterile

MADHU INSTRUMENTS PVT. LTD.

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.
 Regd. Add.: F-90/3D, Okhla Industrial Area Phase-I, New Delhi-110020 India
 Customer Care No.: +91-11-42701030, 42701031
 Email: customercare@madhuinstruments.com
 Website: www.madhuinstruments.com

EC REP

Obelis s.a.
 Bd. Général Wahis 53
 B-1030 Brussels, Belgium
 Phone: 32.2.732.59.54, Fax: 32.2.732.60.03
 E-mail: mail@obelis.net