

EN (English)

# Lacri Care

## Lacrimal Cannula (Lacrimal Intubation Set)

### APPLICABLE FOR THE MODELS

(DCR)Straight/Angled (4.5cm x 20G)

Regular / Olive Tip (7.5, 11, 15, 17.5cm x 23G)

### DEVICE DESCRIPTION

This device consists of two stainless steel malleable probes. (with or without olive tip) attached with a silicone tube.

### INTENDED USE / INDICATION FOR USE

A Sterile single use Lacrimal Cannula (Lacrimal Intubation Set) is used for the treatment of canaliculus system stenosis or nasolacrimal duct obstruction.

### MATERIALS USED

Silicone, Stainless steel

### TARGET AGE GROUP

Can be used for all age groups.

### INTENDED USER

Any of the registered medical doctor/medical professionals.

### DURATION OF USE

Long term use (more than 30 days.).

### STORAGE CONDITIONS

Store at specified temperature and relative humidity (Temperature in between 10°C to 30°C and Relative Humidity in between 45% to 65%).

### CONTRAINDICATIONS


The device should not be used in patients who show hypersensitivity to any of the materials used in manufacturing of the device.

### HOW SUPPLIED

Lacrimal Cannula (Lacrimal Intubation Set) is supplied sterile, packed in two peel open pouch, in such a way which maintain the double sterile barrier system. Such packed pouches are enclosed within an outer printed box. The overall packing contains Lacrimal Cannula (Lacrimal Intubation Set) and medical literature inserted in an outer enclosing box, with external identification stickers.

### CAUTION / WARNING

- Do not use if package is damaged and consult instructions for use.
- Do not use the device if the sterilized package is open or damaged.
- The product is designed for single use only.
- If re use it may lead to cross contamination.

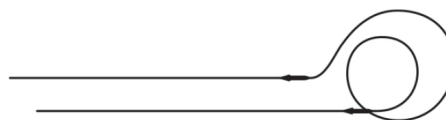

- The use of this product is restricted to a qualified doctor or medical practitioners.
- Store at specified temperature and humidity.
- Keep out of reach of children.
- Reusing the device may cause the transfer of contagious and communicable disease to the patient and other serious problems to patients due to re-use of device such as HIV, Hepatitis, contagious diseases/communicable disease/ undue diseases etc.
- Do not re-sterilize the device by any method. Madhu Instruments assumes no liability for the devices that have been re-sterilized.
- Do not use the device after sterility expiry date.
- Do not pull the silicone tube out of the stainless-steel probe.

### PRECAUTIONS

- Before use, please make sure that the product or packaging has not been tampered or damaged.
- When handling the product care should be taken to avoid damage from handling.

### DIRECTION FOR USE

- Check the package sealing integrity to ensure sterility.
- Select the suitable size of Lacrimal Cannula (Lacrimal Intubation Set) and inspect visually.
- Remove the cannula from sterile packaging.
- Use the Lacrimal Cannula (Lacrimal Intubation Set) as per the standard method used by the surgeons.

### EXPIRATION DATE

The expiration date of the product is the sterility expiration date. The product should not be used after the indicated sterility expiration date, the shelf life for sterile products is recommended to 5 years from the date of sterilization.

### RETURN OF DAMAGED PRODUCT

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/exchange

### REPORTING

Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product related and that were not previously expected in nature, severity or incidence must be reported to Madhu Instruments Pvt. Ltd.

### DISPOSAL OF THE USED DEVICE

The used device should be disposed off in compliance with applicable local, state and country's laws and regulations.

### EXPLICATION OF SYMBOLS:

Symbol	Meaning
	Caution
	Manufacturer
	Date and Country of Manufacture
	Consult instructions for use
	Keep away from sunlight
	Medical Device
	Do not use hook
	This side up
	Temperature Limit
	Humidity Limitation
	Keep Out of reach of children
	Handle with care

Symbol	Meaning
	Maximum stacking
	Batch Code
	Use by date
	Do not use if package is damaged and consult Instructions for use
	Do not re-use
	Single Sterile Barrier System, sterilized using Ethylene Oxide
	Double Sterile Barrier System with protective packaging outside, sterilized using Ethylene Oxide
	Double Sterile Barrier System, sterilized using Ethylene Oxide
	Do not re-sterilize
	Catalogue Number
	Unique device identifier



**MADHU INSTRUMENTS PVT. LTD.**

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.

Regd. Add.: F-90/3D, Okhla Industrial Area Phase-I, New Delhi-110020 India

Customer Care No.: +91-11-42701030, 42701031

Email: [customer@madhuinstruments.com](mailto:customer@madhuinstruments.com)

Website: [www.madhuinstruments.com](http://www.madhuinstruments.com)

Doc. No. MIPL/IFU/E5/03, Issue No. 01, Issue Date: 07.06.2024, Rev.00, Rev. Date: Nil